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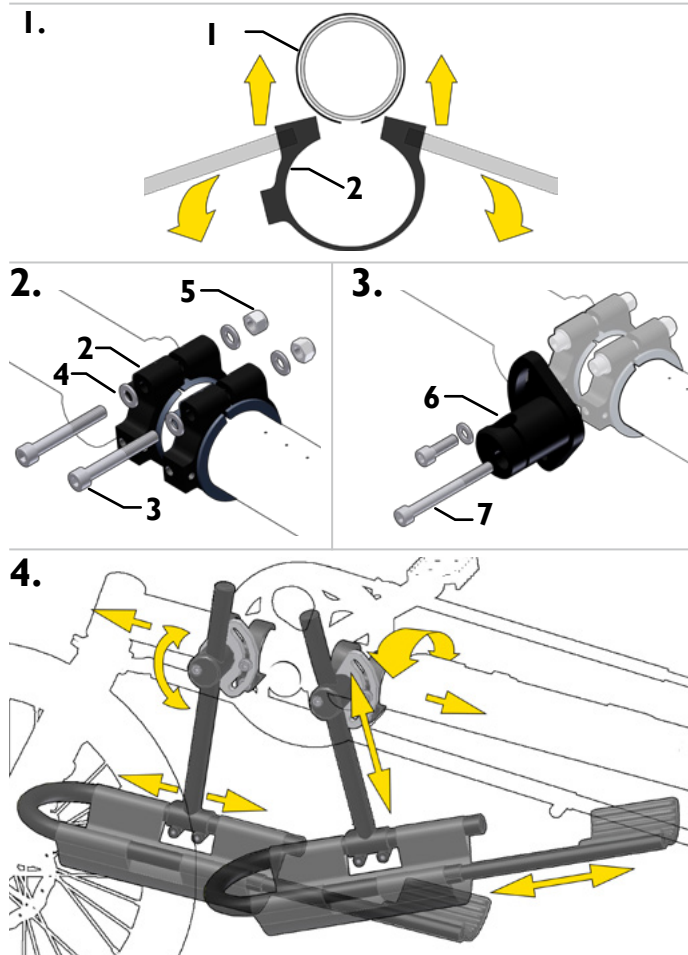
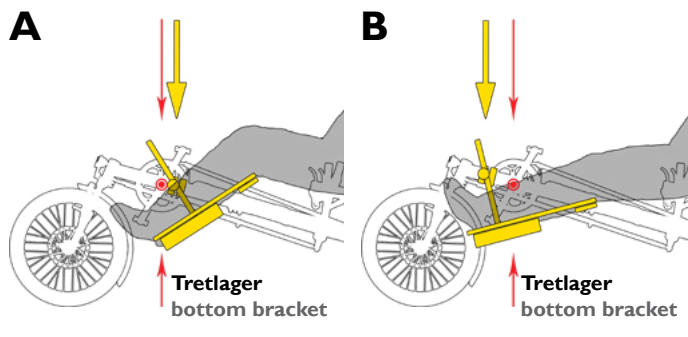
The leg rest fits our model KETTWIESEL as well as the LEPUS and can be used for both sides equally.

The assembly includes the pre-assembled leg-rest and the fitting mount. Furthermore four adapter pieces for mounting on the LEPUS, which are used instead of the slide bearings. Previous to the assembly you have to make sure which kind of permanent leg position you want to achieve (tilted or straight leg)

For attachment you will need following tools:

- allen key (size 4,6,8), wrench (size 13), puller for crank handle
- hacksaw and metal-file (for assembly on the right)
- chain lock + riveter (for assembly on the right with tilted leg position)

Assembly for left side KETTWIESEL

A: rear bottom bracket (tilted leg position)

B: ahead of bottom bracket (straight leg position)

1. Remove the left crank handle with the allen key and the puller.
2. Disassemble the mount of the leg rest. Place the slide bearings(1) by bending around the main pipe behind/ in front of the bottom bracket. Carefully repeat this process with the joint plates(2) (via allen keys or comparable tools, fig. 1). Mind that the grip-spaces should face upward and the lateral openings to the preferred side for the rest. Push the slide bearings under the joint plates and hand-tighten them (with aid of grippers) by using one allen bolt(3), two washers(4) and a lock nut(5) for each joint plate(2). (fig. 2)
3. Mount the grip-cylinder(6) as shown in picture 3. **Tipp: To give additional adjustability, the bended oval hole can be attached facing forward or backward.** Hand-tighten all bolts except bolt(7) (leave it loose).
4. Make sure that the leg rest's assembly is fitting for your desired side (picture 4: left) . If applicable change the assembly. Slide the bar of the leg-rest into the grip-cylinder. Now hand-tighten bolt(7) too.

Adjustment

Picture 4: First adjust the length of the KETTWIESEL's front end to fit the drivers proportions. Then set up the different hinge-angles and distances of the leg-rest. Afterwards thighten all nuts and bolts. (M6: 9Nm, M8: 23 Nm)

Attention: Please note that there must not be any touching point between the rest and the fender or the steering rod when on full lock. We recommend a minimal space of 3 cm to the fender.

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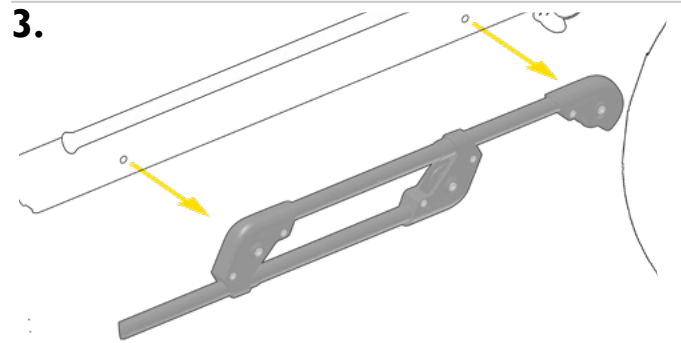
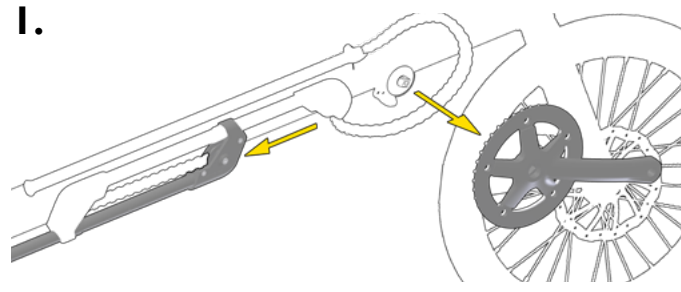
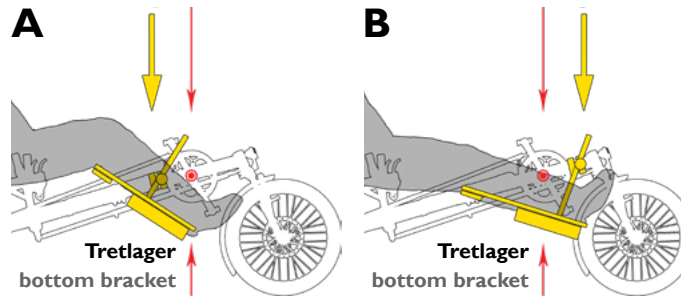
Assembly for right side KETTWIESEL

A: rear bottom bracket (tilted leg position)
B: ahead of bottom bracket (straight leg position)

Attention: For fitting the leg rest on the right side you have to saw off the right crank handle.

A: To achieve a tilted leg position, you have to remove the frame quick-adjustment and shorten the chain (only KETTWIESEL). After that, frame length-adjustment without changing the chain length won't be possible anymore.

- Unscrew the plastic front mount of the quick frame-adjustment. Loosen the right crank with use of a puller. Also loosen the chain by pushing the lower part of the adjustment backwards and lift the chain over the crank handle. (Fig. 1)
- Cut the crank handles as seen in fig. 2 (Place the cut in the highlighted area) and afterwards deburr the edges with a file. Re-mount the remaining gear.
- (For position A only:** Open the chain and remove the quick frame-adjustment (Fig. 3))
- Proceed by following steps 2,3 and 4 of the “assembly for left side”. (Turn the page)

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One-Sided Leg Support

Intended use

The One-Sided Leg Support is used for supporting the leg of a rider with hemiplegia, leg stiffness, or similar health issues.

Contraindications

The Leg Support is not suitable for any purposes that are contrary to its intended use or that could be hazardous. Please observe the contraindications specified in the user manuals of the recumbent bikes and trikes.

Safety Guidelines

When getting out of or into the seat, never put your weight on the Leg Support. Otherwise it could break. Defective or damaged Leg Supports may not be used and must be immediately replaced.

Reuse

Under normal conditions of use, this accessory can be reused by a new owner after inspection by an authorized dealer.

Protection against environmental influences

The Leg Support is made with corrosion-resistant materials. Powder-coated and anodized aluminum parts, plastics, and stainless steel are used.

Cleaning and disinfection

The Leg Support can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

This is an accessory within the meaning of Regulation 2017/745 on medical devices, which as such is CE-marked.

Information regarding incidents

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

* A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

Whom to contact with questions or problems

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

Residual risks

None known

Service life

Depending on the frequency of use, a service life of up to 8 years can be expected if the product is used in accordance with the user manual.

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).